


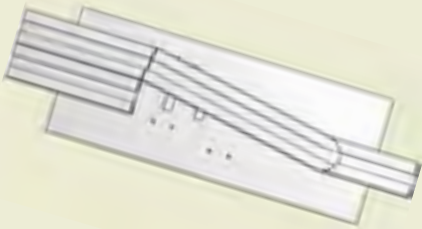

Airglide 360

Track Switch User Manual

General operation of the track switch



1. Move hoist unit in front of track switch until lights glow
2. Wait while the lights are on for 2 seconds until the track switch changes
3. Once the track switch has finished changing, move on through to desired location
4. The track switch will stay in this position until the hoist waits in front of it again for 2 seconds



VERY IMPORTANT!

- Never attempt to move onto the track switch whilst it is moving
- Hoist must pass through the track switch in one movement



IMPORTANT!

- The track switch, hoist unit, and track system should be checked and/or serviced every 6 months. Please contact your Innova™ Dealer for more information.

Questions?

All questions should be directed to your Innova™ Dealer:



For general enquiries please contact us at:

t | 0845 034 1450

f | 0845 034 1451

e | enquiries@innova.uk.com

www.innova.uk.com