

/ Somlent Genius Mattress

Quick User Guide

POWER


On/off button. Keep finger on button to turn the pump off.

SENSOR RESET

Press these 2 buttons together for 1 second to reset the sensors.

PANEL LOCK

Press these 2 buttons together for 1 second to unlock the panel before pressing any other buttons.

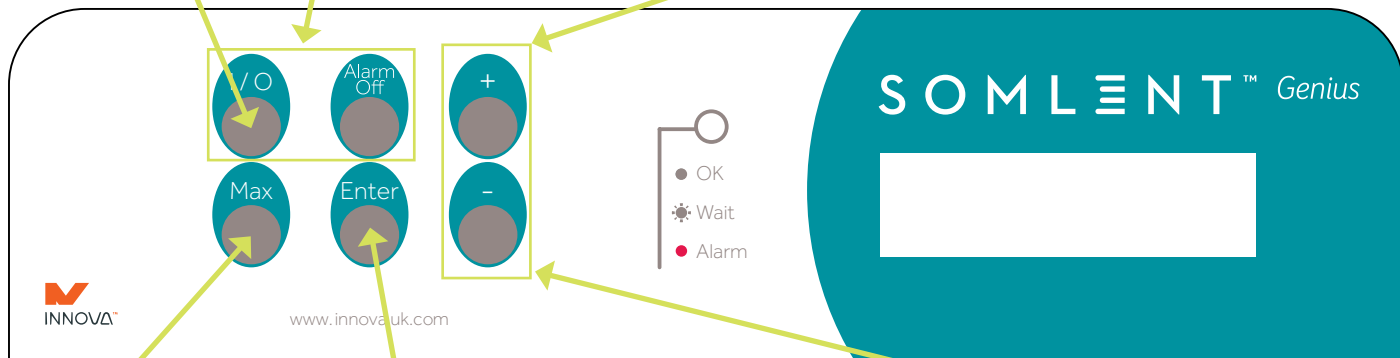

MAX INFLATE

Press this button to inflate the mattress to maximum. Used for patient transfers.

ENTER BUTTON

Press this button for 5 seconds to change the comfort level of the head, torso, and foot sections separately.

COMFORT SETTINGS

Once panel is unlocked, comfort of the mattress can be made harder (+ button) or softer (- button) with a range of -5 to +15.